

LECKEY®

Everyday
Activity Seat



MONDAY
TUESDAY
WEDNESDAY
THURSDAY
FRIDAY
SATURDAY
SUNDAY



EVERYDAY

LECKEY^o

Stronger together.

We work with individuals, therapists and carers to design products with both a clinical and an emotional focus. Using the latest research and clinical understanding, we create practical solutions which are easily integrated into family life, because for us, life is about going, enjoying, participating and doing.



EVERYDAY
SEATING I
EVERYDAY
EXPERIEN

24 hour
postural care for
babies, kids & adults.
Sleeping, Sitting,
Standing, Walking,
Moving, Bathing,
Toileting.




FOR
VICES

WHETHER YOU'RE
READING, LEARNING,
COMMUNICATING OR
PLAYING... IT'S ALL
IN A DAY'S WORK
WITH THE EVERYDAY
ACTIVITY SEAT.



FOR KIDS
AGED 1-18
WITH
MODERATE
POSTURAL
NEEDS

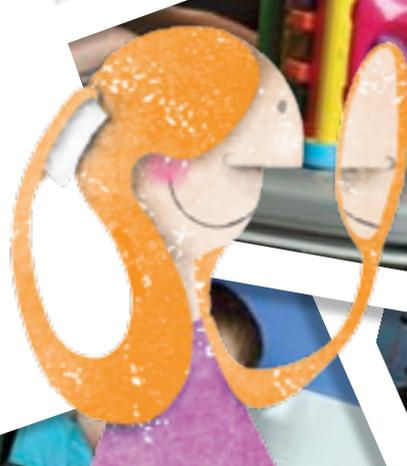



KEY FEATURES & BENEFITS



1 The options available – 3 sizes,* 2 chassis, 3 colours and a range of accessories – allow the therapist to exactly match the client's postural and practical needs.

2

Provides an optimal level of postural support for children with moderate needs so that they can enjoy activities at home, school or in respite.





For people of all ages and abilities, postural stability is the key to performing everyday tasks such as eating, writing, or using a computer – anything which requires co-ordinated motor control. That's why Leckey has developed the Everyday Activity Seat, specifically for children with moderate needs. These children do not need the same advanced level of postural support as those with complex needs, but they still require pelvic stability to perform everyday tasks to the best of their abilities. And stability is what the Everyday Activity Seat provides.



3 The seat has been designed to be easy to adjust and care for. The brightly coloured covers can be removed and machine washed.

4 The wide range of height adjustment on the hi-low base means that it goes low enough for standing transfers or high enough for sitting at a school desk or home dining table.

* Size 3 available in 2013.

PLAYING
COMPUTER
GAMES AFTER
SCHOOL!

EVERY SEATING EVERY EXPERI

SUNDAY DINNER
WITH THE FOLKS!

MATHS & A
VIEW WITH
TES ON
TODAY




EVERYDAY SEATING FOR EVERYDAY EXPERIENCES



EVERYDAY SEATING FOR EVERYDAY EXPERIENCES

BOARD GAMES ON A RAINY THURSDAY

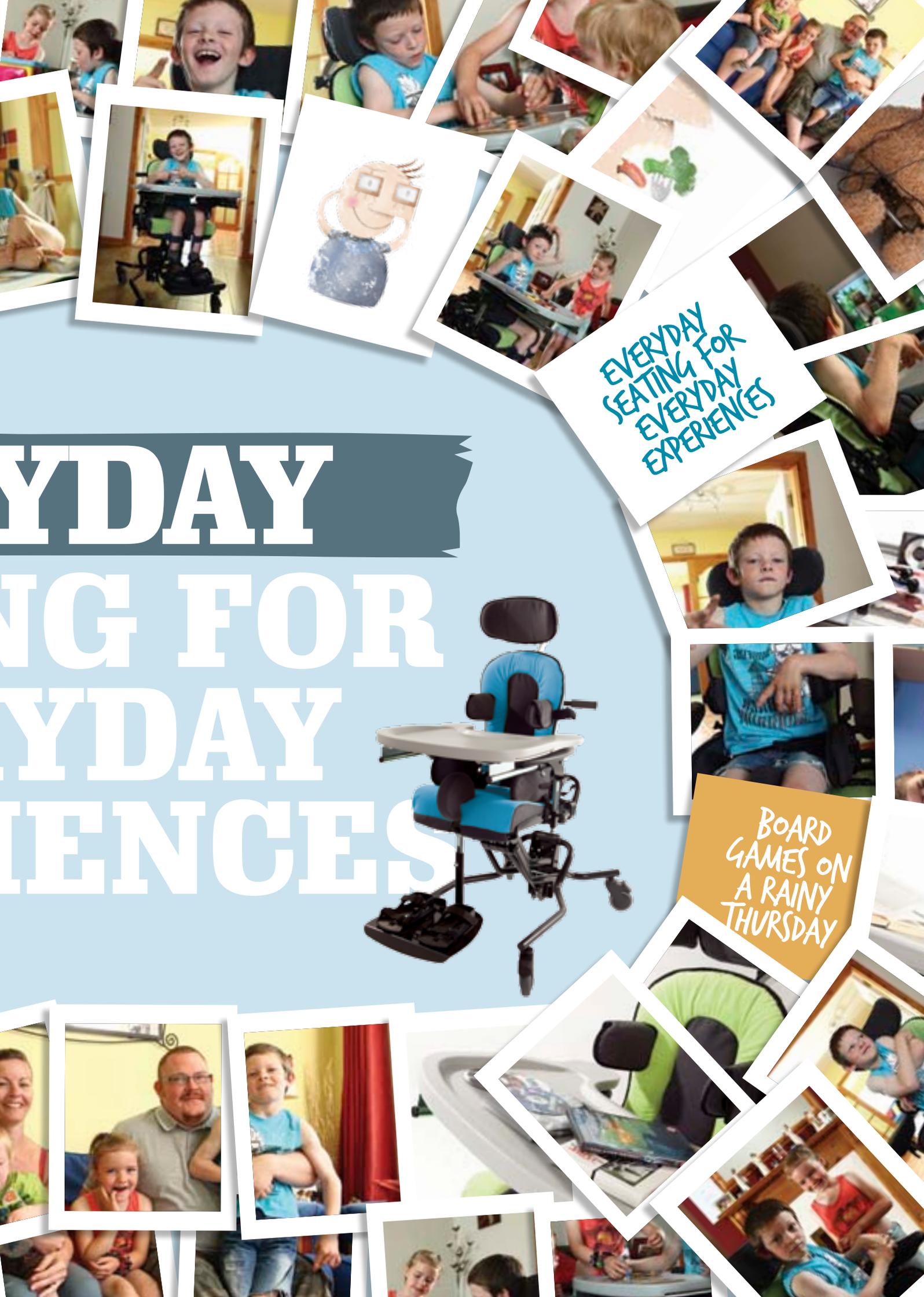



1. Cushioned headrest with depth and height adjustment has removable, machine washable covers.
2. Cushion covers come in three bright colours and can be removed and machine washed. The seat base cover tucks under the front of the seat for added comfort.
3. A range of height and angle adjustable chest laterals are available. Standard, flip-away or complex laterals can be used to provide appropriate support for the trunk.
4. The 4 point pelvic harness gives a secure, stable base of support, is cushioned for comfort and can be adjusted to ensure a mid-line position.
5. The height and angle adjustable armrests are made of soft PU. They have been designed around the tray receivers with the client's comfort in mind.
6. The soft, upholstered pommel provides abduction if required.
7. The sandals can be angle and depth adjusted on the footplate. Sandal raisers (not shown) are also available.
8. The height and angle adjustable footplate can be flipped up to facilitate transfer. It is available in metal or wood (not shown).
9. The manoeuvrable lockable 100mm castors move freely over flooring in the home, school or therapy environments.
10. The foot pedal operated hi-low chassis has a large range of height adjustment allowing the client to be seated low, for example during circle time at school, or high, for example helping in the kitchen. Please note that the seat cannot be removed from the chassis.
11. The one-piece seat base is depth adjustable. The width can be adjusted using the side pads. The soft, comfortable cushion is ramped and has a removable cover.
12. The soft, upholstered side pads provide width adjustment for the seat base as well as additional lateral stability.
13. The tilt-in-space angle can be adjusted with the client in the seat. This angle adjustment can support various positions, whether it be a supine position for resting or a prone position to aid a standing transfer.
14. The small, unobtrusive push handles allow the seat to be moved around easily.
15. The backrest can be height and angle adjusted. The back angle can be opened up for resting, for example after a meal, or can be closed to promote trunk and head control. Extra padding can be added at the sacral area to promote a neutral or anterior pelvic position.
16. The activity tray can be positioned according to the child's needs. The height can be adjusted, for example to elbow height for improved function or higher for increased anterior support for low tone or fatigued children. The depth adjustment makes it suitable for different builds of client and allows the tray to be placed snug against the body while eating. The tray is available in PU or wood (not shown).
17. The height range of the basic chassis is suitable for clients who do not need the seat at floor level for activities or transfer. Please note that the height adjustment cannot be done with the client in the seat.



EVERYDAY ELEMENTS



THREE CLINICAL ADVANTAGES OF USING THE EVERYDAY ACTIVITY SEAT


OPTIMAL POSTURE + STABILITY EVERYDAY



Additional pelvic support benefits children with moderate postural control needs because the pelvis provides the key to overall postural stability. The Everyday Activity Seat offers exceptional pelvic positioning with a four-point pelvic harness.

Complemented by the seat's simple range of trunk, head, leg and foot positioning adjustments in keeping with the needs of this group of children, the Everyday Activity Seat provides optimal postural stability.

OPTIMAL COMFORT EVERYDAY

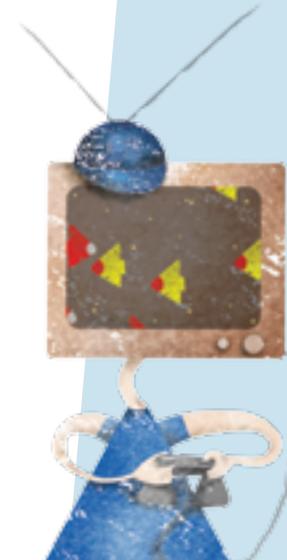



OPTIMAL COMFORT EVERYDAY

It is not possible to sustain an optimal posture without comfort because discomfort disturbs posture. The Everyday Activity Seat is designed with comfort of the young person in mind in every aspect. From the soft, breathable covers, to the moulded armrests, and the clever seat base cover which tucks underneath the seat base, the Everyday Activity Seat ensures the comfort (and therefore postural stability) of the user.



1. Hi-low chassis at floor level to allow little brother to join in.



2. Mid-height in line with the TV to play with the Wii.



3. Meal-time: table height with armrests lowered.

OPTIMAL FUNCTION EVERYDAY



4. Full height to reach the sink for hand washing.

Function (the ability to do everyday things) depends on the combination of postural stability and comfort. With optimal postural stability and comfort achieved, children are able to use their arms and hands more effectively and independently.

Along with the Everyday Activity Seat's height adjustability from floor to table (hi-low chassis), activities such as playing, eating, reading, writing or chatting with friends become easier and less tiring.



EVERYDAY SIZES



Size	1		2		3*	
Age	1-6		5-12		11-18	
	mm/kgs	inches/lbs	mm/kgs	inches/lbs	mm/kgs	inches/lbs
Max load	30kgs	66lbs	50kg	110lbs	100Kgs	220lbs
Weight - seat and basic chassis	22.5kg	50lbs	27kg	60lbs	29kg	64lbs
Weight - seat and hi-low chassis	29.5kg	65lbs	34kg	75lbs	n/a	
Seat height - basic chassis	450-600	17.7-23.6	450-600	17.7-23.6	450-600	17.7-23.6
Seat height - hi-low chassis	250-600	9.8-23.6	320-600	12.6-23.6	n/a	
Seat width	160-280	6.3-11	260-380	10.2-15	340-440	13.4-17.3
Seat depth	150-280	5.9-11	230-380	9-15	330-480	13-18.9
Lateral distance	150-220	5.9-8.7	170-240	6.7-9.4	240-300	9.4-11.8
Back height	300-400	11.8-15.7	400-520	15.7-20.5	450-600	17.7-23.6
Back angle	30° supine 10° prone		30° supine 10° prone		30° supine 10° prone	
Armrest height	150-250	5.9-9.8	150-250	5.9-9.8	180-325	7-12.7
Lower leg length - basic chassis	150-250	5.9-9.8	280-450	11-17.7	280-450	11-17.7
Lower leg length - hi-low chassis	150-250	5.9-9.8	280-450	11-17.7	n/a	
Lower leg angle	-10° to +50°		-10° to +50°		-10° to +50°	
Footplate angle	-15° to +10°		-15° to +10°		-15° to +10°	
Tilt-in-space	25° supine 10° prone		25° supine 10° prone		25° supine 10° prone	

* Available in 2013

Seat shell and chassis

The standard product includes one piece seat base, split backrest with push handle and chassis with gas spring (hi-low chassis only) and 100mm castors.

Everyday Activity Seat on basic chassis



Everyday Activity Seat on hi-low chassis



Seat shell on basic chassis




Seat shell on hi-low chassis


Cover pack

3 Colour Options


Each cover pack will include the following:
Seat base cushion & cover
Backrest cushion & cover
Sacral pad inserts

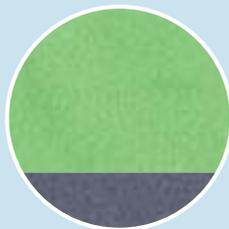

Green & Grey

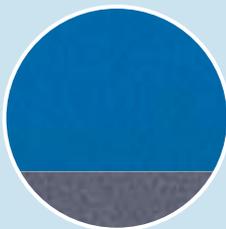

Blue & Grey

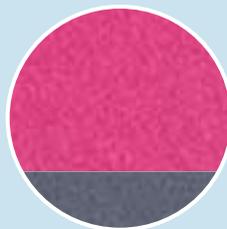

Pink & Grey


Servicing

For service policies on all products outside warranty, please contact your Customer Service department. James Leckey Design Ltd as manufacturer with sole responsibility declares that all products conform to 93/42/EEC guidelines and EN12182 technical aids for disabled persons general requirements and test methods. Order forms and spare parts lists to extend the service life of the product and allow reissue are available on request.

EVERYDAY ACCESSORIES



Contoured headrest hardware and grey cover

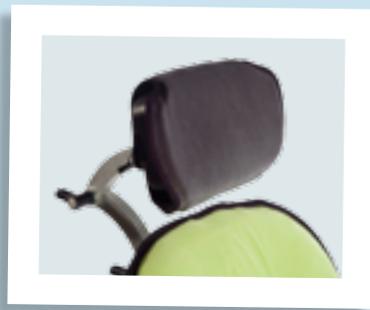

Flat headrest hardware and grey cover



Flat headrest hardware and grey cover (incl laterals)



Rigid PU chest laterals (grey cover included)



Flip-away PU chest laterals (grey cover included)



Basic chest laterals (grey cover included)



Complex flip-away chest laterals



Protraction pads



Armrests



Chest harness
(grey cover included)




Trunk harness (black)




Trunk harness extension straps




Side pads (pair, shown with
grey covers)

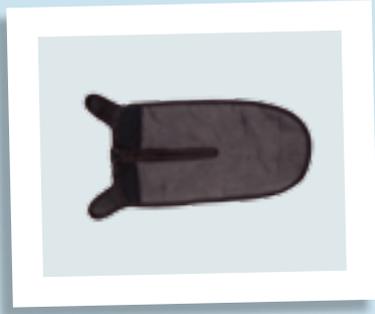

Side pad grey covers (pair)

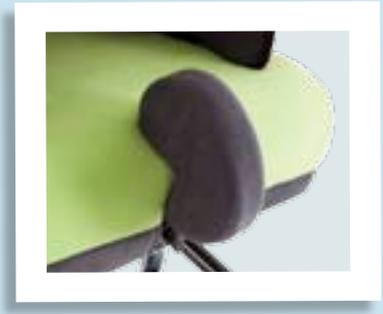

Pommel

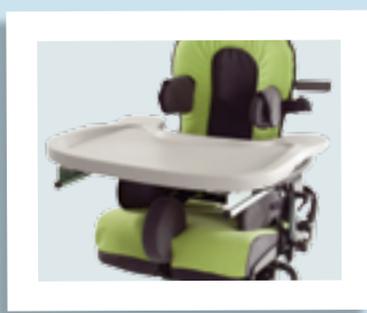

Activity tray (plastic)



Activity tray (wood)



4-point pelvic harness



Footplate (metal)



Footplate (wood)



Sandals, incl straps
Sandal raiser (single, not shown)

LECKEY^o


Established in 1983, Leckey is a globally recognised pioneer in the research and development of products that help adults and children with disabilities to go, do, enjoy and participate in everyday activities throughout the day and night.

We take a highly clinical approach to product design and development. Through in-depth clinical research studies with leading universities, and extensive trials with occupational therapists, physiotherapists, users and their families, we continue to develop posturally supportive, family friendly products for all day care, at every stage of life.

Through early intervention, childhood and adulthood, Leckey's experienced team of designers, therapists and bioengineers work together to develop products that meet the clinical needs of the healthcare professionals and the social needs of the user.

To achieve this, we work with the healthcare professionals, the individuals and carers who use our products every day. With their help, we create the dependable, durable, proven and high performance products that we are known for worldwide.



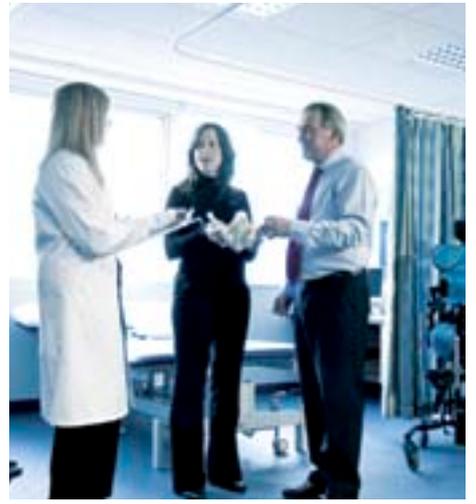

Manufacturer

Leckey
19 Ballinderry Road
Lisburn
BT28 2SA
Northern Ireland
United Kingdom

T: 028 9260 0750
F: 028 9260 0799
E: info@leckey.com
W: www.leckey.com

Distributor

Otto Bock
HealthCare GmbH
Max-Näder-Straße 15
37115 Duderstadt
Germany

T: 0049 5527 848-1590
F: 0049 5527 848-1676
E: reha-export@ottobock.de
W: www.ottobock.com

24 hour postural
care for babies,
kids & adults.
Sleeping, Sitting,
Standing, Walking,
Moving, Bathing,
Toileting.