Liner Guide

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EN First Steps

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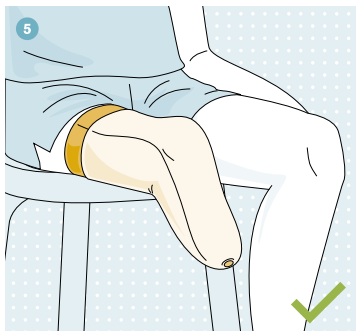
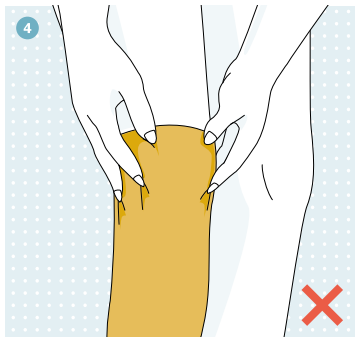
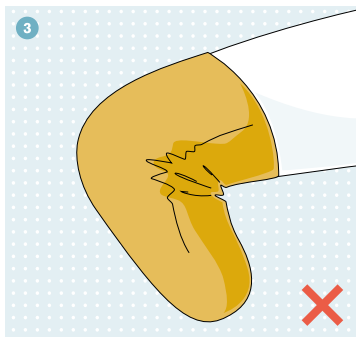
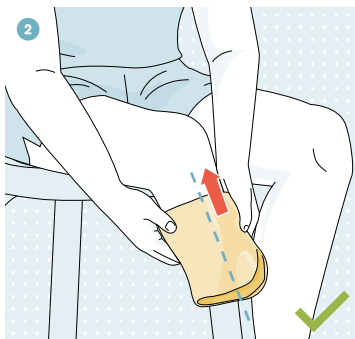
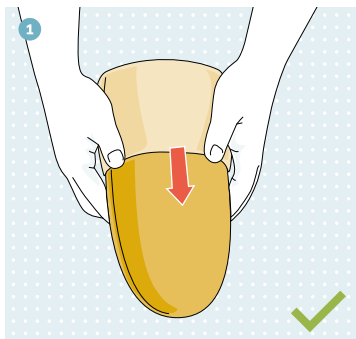
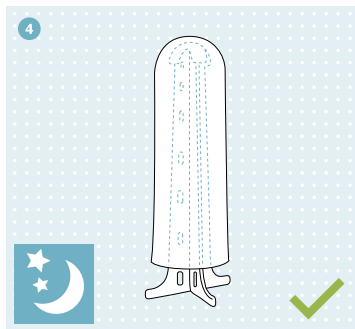
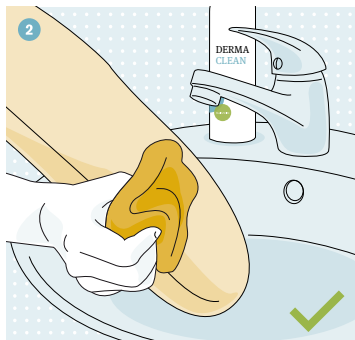
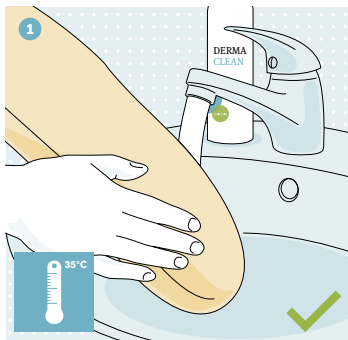
Quality for life

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Donning and doffing the liner

- **Do not wear the liner on injured skin and wash it before using it for the first time.**
- **Liners without a textile cover have a smooth exterior coating to make donning easier. If necessary, donning spray for silicone liners or a lubricating cream for PUR liners can be applied additionally. Do not use any powders.**

- 1** Turn the Ottobock liner inside out as far as possible and hold it such that the bulge at the lower end (cap) is visible. Please check whether the residual limb and inside of the liner are dry and free of dirt particles.
- 2** Position the closed end of the liner parallel to the residual limb end and roll up the liner evenly and carefully.
- ▶** If there is a pin, this must point in the direction of the lengthwise axis of the residual limb. For better alignment, use markings for orientation that can be determined with your prosthetist.
- 3** Caution! Avoid soft tissue displacement, the formation of wrinkles and air inclusions.
- 4** Do not pull on the edge of the liner or damage the surface with sharp fingernails or sharp objects.
- 5** Pull over the residual limb sock in such a way that the pin, if applicable, remains free.
- 6** When unrolling the liner during doffing, make sure that the edge of the liner does not roll up into a pressing bulge.

Caring for your liner

- Clean the liner after each use and at least once daily

- ▶ Do not put the liner in the washing machine or dryer.
- 1 Turn the liner inside out and wash it using lukewarm water and the neutral Ottobock special cleaning lotion DERMA Clean. Rinse thoroughly. Use only your hands and no other utensils to wash the liner.
 - ▶ Do not use any soaps with perfume or dye additives or alcoholic cleaning agents with an antibacterial action. These could damage the liner material and cause skin irritation.
 - 2 Make sure that no soap residue remains and then dry the inside surface of the liner with a lint-free cloth. When doing this, check the liner for wear and damage. If you notice any changes, contact your medical supply company.
 - 3 Turn the liner back the right way round. Clean the outside likewise under a tap or with a damp cloth.
 - 4 Leave the liner to dry (if possible overnight) in its correct shape on the liner drying stand. The liner stand is available in a small size (5Y100=1 for transtibial liners) and a large version (5Y100=2 for transfemoral liners + 6Y93F).
 - ▶ Please note that you should NEVER leave the liner to dry on a radiator.

Skin care

- Use care products only externally and not on open wounds or infections; avoid eye contact
- See the product packaging for information on the contents

Wearing a prosthesis can lead to the natural function of the skin being compromised, leading to the formation of pressure or friction points. Incorrect hygiene can also contribute to this. Therefore, protect your skin with the DERMA skin care range from Ottobock developed especially for these requirements.

453H10 Derma Clean: Mild and reliable cleaning

The special cleaning lotion can be used for cleaning liners (see "Caring for your liner") and your skin. To do this, spray a small amount directly onto your skin and rub it in. Rinse well afterwards and dry.

- PH-neutral, free of alkali and phosphates
- Antibacterial formula

453H14 Derma Repair: Fast aid for stressed skin

Apply this special basic skincare product over a large area of the skin in the evenings after the daily cleaning.

- For noticeably moister and more elastic skin
- Improves blood circulation and actively supports regeneration and cell growth
- Calms irritated skin and makes it more resilient

453H12 Derma Prevent: Protective and preventative action

Apply a few drops of the special skin protection to the palms of your hands and rub onto your skin.

- ▶ Please do not use if you are wearing a liner made from polyurethane or copolymer.
- Prevents wounds and excessive perspiration and odour formation
- Protects skin from irritation and keeps it supple
- Inhibits contact with external allergens and has an antibacterial effect

FAQ

Question: I have purchased a new liner and have noticed increased formation of sweat for a few weeks. What can I do to prevent this?

In the first weeks, heavy perspiration may occur. Therefore, the liner should be cleaned several times a day. It is also advisable to have a replacement liner at the ready.

If these measures do not bring a positive change, there are various causes that must be considered. A poor fit, the inclusion of deep scars or donning the liner incorrectly can all lead to air inclusions, which can promote heavy perspiration. Another main reason can be the wrong choice of liner material. Ask your prosthetist for advice, so that your individual causes can be determined and your liner will not be damaged.

Question: There are red stripes on my skin at the level of the edge of the liner. What is the reason for this and what can I do against it?

Red stripes mostly appear in connection with silicone liners. Silicone has high adhesive properties on the skin. This can appear particularly in areas where the liner moves a lot on the skin, e.g. on the kneecap or at the edge of the liner.

If you have similar problems, please contact your prosthetist.

Question: My liner has the SKINGUARD antibacterial additive. Does this mean that I do not have to clean the liner as often?

The SKINGUARD antibacterial additive from Ottobock does indeed protect the liner from bacteria and unpleasant odours resulting from bacteria. The effectiveness of the treatment with SKINGUARD has been proven using internationally recognised microbiology test procedures. In particular, it helps your hygiene, which cannot be achieved alone through washing the liner. Nevertheless, the treatment with SKINGUARD does not replace the need for liner care. The liner must still be cleaned at least once a day (as described in the "Caring for your liner" section of this leaflet) to ensure permanent hygiene. This is the only way to remove grease and oil, for example.

Question: Vapomats are available on the market for cleaning liners. Can I also use one for my Ottobock liner?

Ottobock liners are made from different materials in order to be able to meet users' individual needs. You should only use a vapomat for Ottobock silicone liners. In any case, contact your prosthetist about your intentions in order to clarify the suitability of this type of cleaning for your liner.

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