

WalkOn[®] Family of AFOs

Product Information



WalkOn
..... Trimable
28U23

WalkOn
Flex 28U22



WalkOn
..... 28U11



Lightweight, Low Profile, Durable Stabilization

Using any of the WalkOn® AFOs results in a gait that is more natural and symmetrical, with fluid roll-over and good energy return. Our posterior design allows for significant plantar and dorsiflexion range and makes it easier for patients to go up and down stairs or inclines naturally. The sleek design features a posterior strut, making it easier to fit and more comfortable than AFOs with an anterior strut.

As light as 6 ounces, the WalkOn family packs maximum function into a low-profile package. Even the calf band goes the extra mile, made of a material that helps control temperature fluctuations, ultimately keeping your patient more comfortable.

Features

- Extremely durable yet very lightweight due to unique material and design
- Provides heel compression at initial contact, and dynamic movement from midstance to terminal stance for easy rollover
- Posterior design allows material to flex, which reduces the risk of material failure

Benefits

- Fits comfortably inside virtually any shoe
- Easy to don and doff
- Open heel design helps to ensure good fit of heel counter

General Indications

Ideal for patients with drop foot, dorsiflexor paresis/paralysis, peroneal nerve palsy or CVA.

28U11 WalkOn & 28U23 WalkOn Trimable

Ideal for patients requiring greater support for moderate to severe drop foot. Fabricated from a prepreg advanced carbon composite, the WalkOn and WalkOn Trimable deliver superb stability in a low-profile package.

28U22 WalkOn Flex Ideal for patients requiring less support for mild drop foot. The posterior spiral strut provides more comfort and flexibility for patients.

Trace, Trim and Walk Away

The WalkOn® Trimable and the WalkOn® Flex offer your choice of support coupled with the simplicity of shaping the footplate with a scissors, allowing you to deliver personalized solutions to your patients wherever they are.

How to Trim and Apply

Adjust the sole:

- 1 If the user has a removable sole in his/her shoes, use that for marking the correct size on the WalkOn Trimable and WalkOn Flex sole (Fig. 1).
- 2 Cut the trimable sole after your markings. The lateral side should not be cut more than is required, so that the medial part of the spring and ankle does not press against the medial aspect of the foot (Fig. 2).
- 3 Place the sole of the WalkOn under the removeable insole, or add a thin padded layer over the sole of the WalkOn.



Efficient patient fittings with Fit Kits

The kits provide everything you need for efficient patient fittings or referral source presentations! The kits, which fit into a convenient shoulder bag, include:

- Two size small (left and right)
- Two size large (left and right)
- Four calf pads



▶ 28T2 WalkOn® Flex Fit Kit



▶ 28T1N WalkOn® Fit Kit

How To Order

WalkOn Article No.	Left/Right	Size	Shoe Size		Height
			US Mens	US Womens	
28U11/28U22/28U23 = L36-39	L	Small	5 – 6	6 – 8½	13.8 inches (35 cm)
28U11/28U22/28U23 = R36-39	R	Small	5 – 6	6 – 8½	13.8 inches (35 cm)
28U11/28U22/28U23 =L39-42	L	Medium	6 – 9	8½ – 10	14.2 inches (36 cm)
28U11/28U22/28U23 =R39-42	R	Medium	6 – 9	8½ – 10	14.2 inches (36 cm)
28U11/28U22/28U23 =L42-45	L	Large	9 – 12	10 – 13	14.8 inches (37.5 cm)
28U11/28U22/28U23 =R42-45	R	Large	9 – 12	10 – 13	14.8 inches (37.5 cm)
28U11/28U22/28U23 =L45-48	L	Extra-Large	12 – 15		15.4 inches (39 cm)
28U11/28U22/28U23 =R45-48	R	Extra-Large	12 – 15		15.4 inches (39 cm)

WalkOn Replacement Straps and Liners

29U30=LL-1	WalkOn Strap & Liner	Left Lateral	S/M
29U30=LL-2	WalkOn Strap & Liner	Left Lateral	L/XL
29U30=RL-1	WalkOn Strap & Liner	Right Lateral	S/M
29U30=RL-2	WalkOn Strap & Liner	Right Lateral	L/XL

To get your patients into a high-quality, WalkOn AFO, call your local Ottobock Sales Representative at 800 328 4058.