Otto Bock®

QUALITY FOR LIFE

## 12K100=RENTAL DynamicArm®

<b>DE</b>	<b>Aufbau- und Montageanleitung</b> .....	2
<b>EN</b>	<b>Alignment and Assembly Instructions</b> .....	11
<b>FR</b>	<b>Notice de montage et d'alignement</b> .....	19
<b>IT</b>	<b>Istruzioni per l'allineamento ed il montaggio</b> .....	27
<b>ES</b>	<b>Instrucciones de alineamiento y montaje</b> .....	38
<b>PT</b>	<b>Instruções de construção e montagem</b> .....	47
<b>NL</b>	<b>Opbouw- en montagehandleiding</b> .....	56
<b>SV</b>	<b>Inriktnings- och monteringsanvisning</b> .....	65
<b>DA</b>	<b>Tilpasning og monteringsinstruks</b> .....	74
<b>NO</b>	<b>Opbygnings- og monteringsvejledning</b> .....	83
<b>EL</b>	<b>Οδηγίες ευθυγράμμισης και συναρμολόγησης</b> .....	92

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---

These instructions for use were last updated on 18/01/2011.

**Before fitting the patient with the 12K100=RENTAL DynamicArm®, please first read the 647G152 Instructions for Use for the 12K100 DynamicArm®!**

## 1 Intended Use

### 1.1 Medical Purpose

The 12K100=RENTAL DynamicArm® is to be used **solely** for the duration of the service and for trial fittings for exoprosthetic fittings of the upper limbs.

### 1.2 Application

The 12K100=RENTAL DynamicArm® is suitable for unilateral and bilateral above-elbow amputees. The 12K100=RENTAL DynamicArm® is to be used **exclusively** for the fitting of adults.

### 1.3 Application Conditions

The 12K100=RENTAL DynamicArm® has been developed for everyday use and must not be used for unusual activities such as extreme sports (free climbing, paragliding, etc.).

### 1.4 Prosthetist Qualification

The fitting of a patient with a 12K100=RENTAL DynamicArm® may only be carried out by prosthetists who have been authorized by Otto Bock based on completion of a corresponding training course.

## 2 Safety Instructions

Please learn the safety instructions as well as patient information from the 647G152 Instructions for Use for the 12K100 DynamicArm®.

Failure to follow the safety instructions mentioned below can lead to faulty control or malfunction of the 12K100=RENTAL DynamicArm® and result in risk of injuries for the patient.

## 3 Service Information


The 12K100=RENTAL DynamicArm® must be sent for examination to the authorized Otto Bock Service after every patient fitting. In the service the complete Rental DynamicArm® will be examined and possibly readjusted and any wearing parts will be exchanged if necessary.

## 4 Adjustment, Fittings, Application

Before fitting the patient with the 12K100=RENTAL DynamicArm®, please first read the 647G152 Instructions for Use for the 12K100 DynamicArm®. They include important additional information about the application of the 12K100=RENTAL DynamicArm®.

## 5 Scope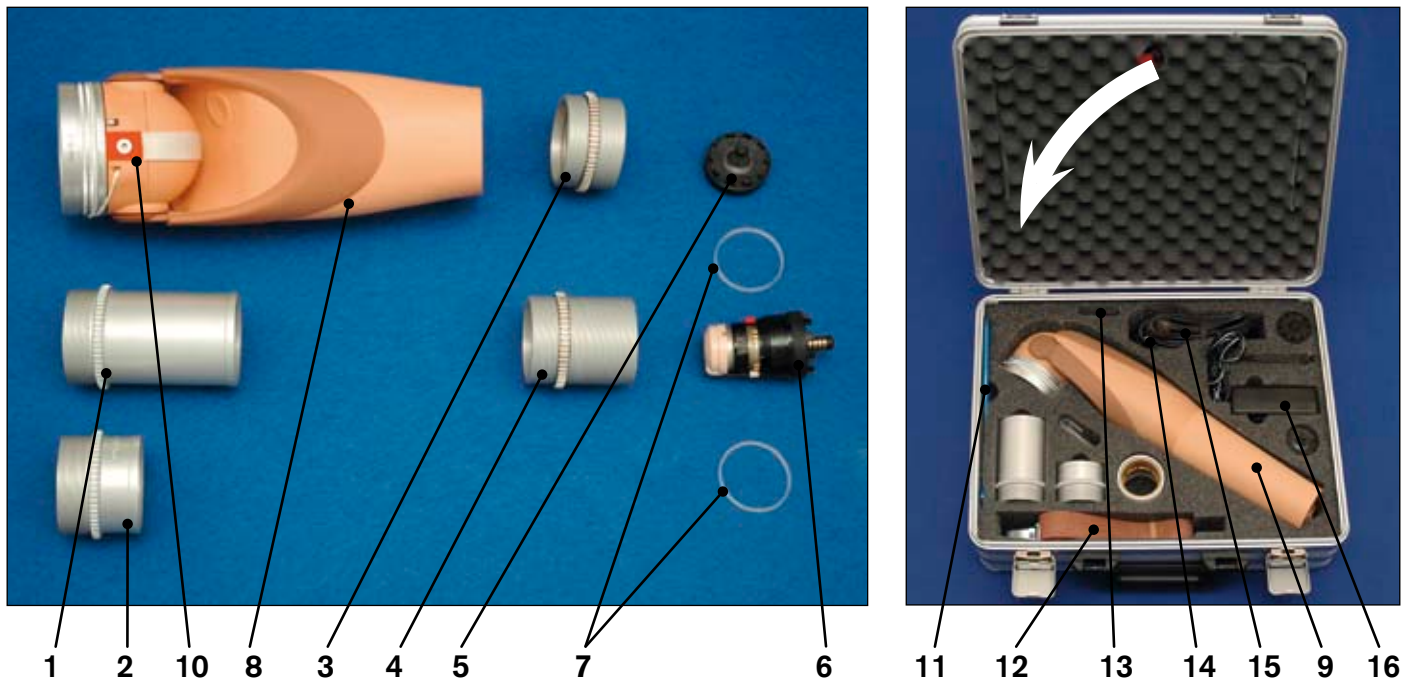

### 5.1 System Components

- Item 1 Adapter long
- Item 2 Adapter short
- Item 3 Threaded tube, short, with lamination ring and counter nut
- Item 4 Threaded tube, long, with lamination ring and counter nut
- Item 5 9E169 Coaxial Plug
- Item 6 10S17 Electric Wrist Rotator
- Item 7 Retaining ring
- Item 8 12K100=RENTAL DynamicArm®
- Item 9 Disposable foam cover
- Item 10 Red assembly clamp

### 5.2 Accessories and Tools

The following items have been supplied additionally to the system components:

- Item 11 Hook wrench
- Item 12 Strap wrench
- Item 13 Suitcase key
- Item 14 Euro mains cable
- Item 15 US mains cable
- Item 16 Battery charger

### 5.3 Manuals

All manuals necessary for the fitting can be found on the inner side of the suitcase under the foam cover.

no illustr. 647G152 Instructions for Use for DynamicArm®

no illustr. 647G370 Alignment and Assembly Instructions for Rental DynamicArm®

no illustr. 647H324 Note tag for reusable package

no illustr. 647H204 Instructions for Use for Electric Wrist Rotator

no illustr. Instructions for Use for Battery Charger

no illustr. 646D299 Patient Information for DynamicArm®


## 6 Disassemble the Patient's Prosthesis

Before adapting the 12K100=RENTAL DynamicArm® to the patient, the patient's elbow harness must be removed from the socket.

**Note !** The 12K100=RENTAL Rental DynamicArm® is compatible with upper arm sockets of the 12K100 DynamicArm® or series 12K44 or 12K50 ErgoArm®.

Example: Prosthetic socket with 12K100 DynamicArm® and system electric hand.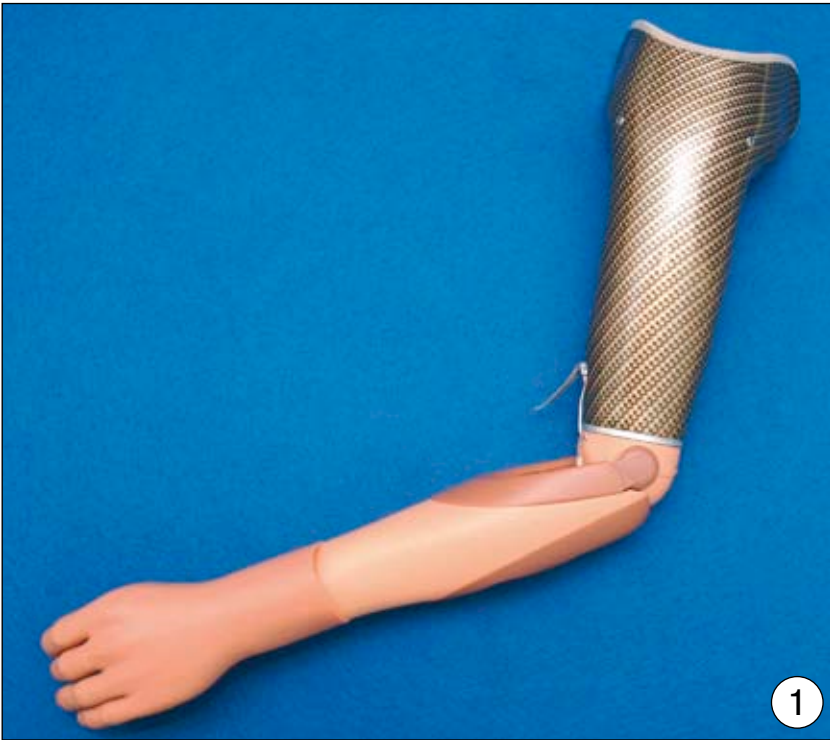

Rotate the prosthetic component (system electric hand or system electric greifer) until the quick disconnect wrist unlocks. Carefully pull off the prosthetic components.


**Caution!** The DynamicArm® must be switched off before removing the prosthetic components.


Turn the finger wheel counter-clockwise to set minimum compensation. This will prevent the band from being pulled in.


**Note:**  
A DynamicArm® with a retracted band must only be repaired by an authorized Otto Bock® Myo-Service provider!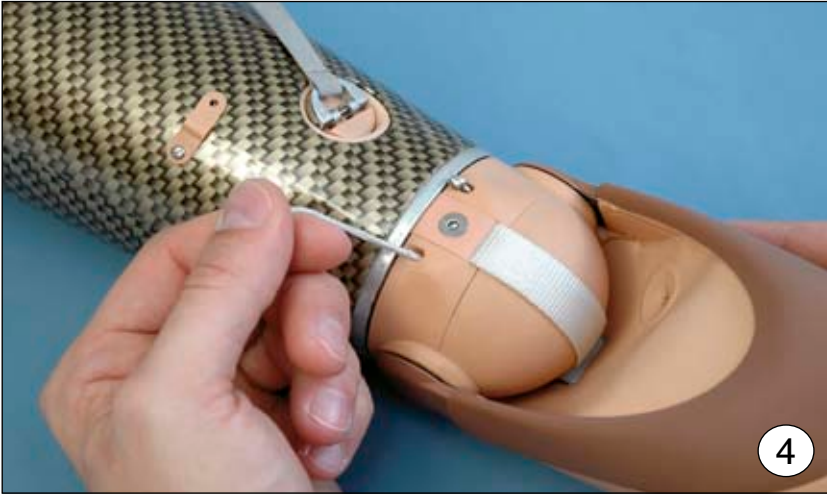

Extend the elbow and lock it with the Bowden cable. Switch off the DynamicArm®.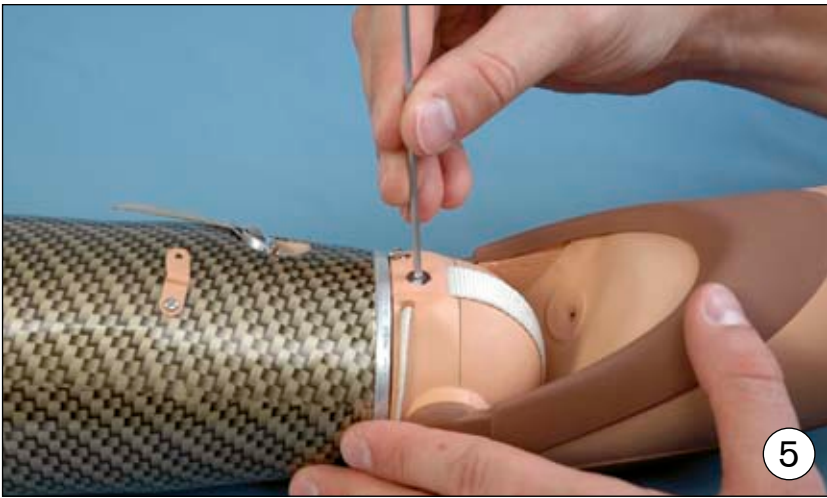

Remove the band clamp.

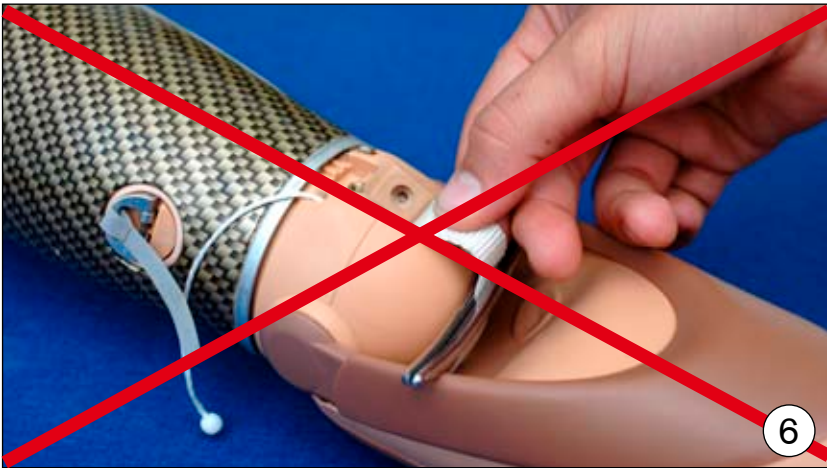
**Attention:** The lifter band should not be removed from the elbow ball. Fixing the lifter strap in place (e.g. with a haemostat) is no longer necessary.


Lift the band clamp out.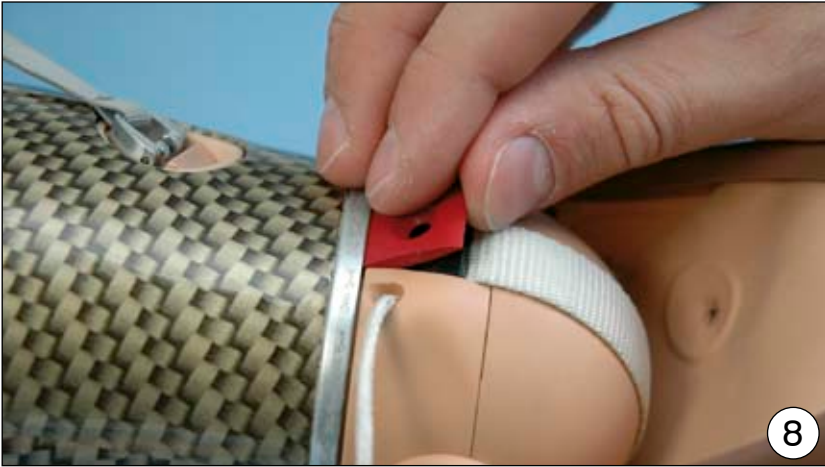


Insert red assembly clamp.

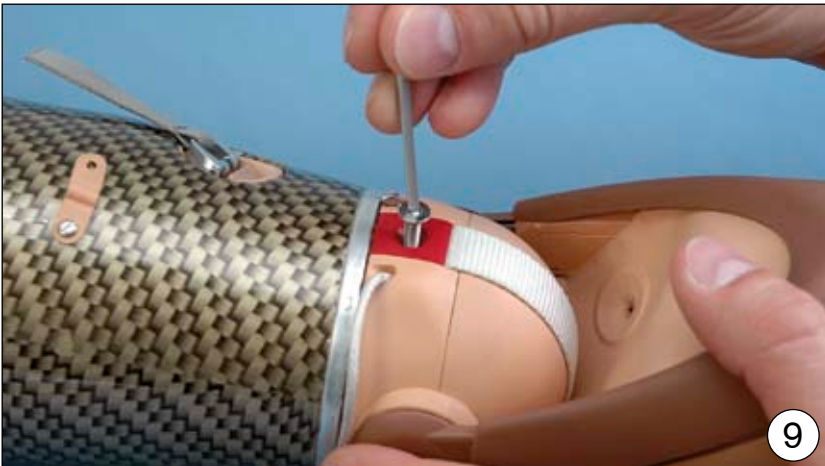Tighten the Allen head screw.

Turn the friction screw until the socket can be screwed off the elbow joint.  
Unscrew the socket from the DynamicArm®.

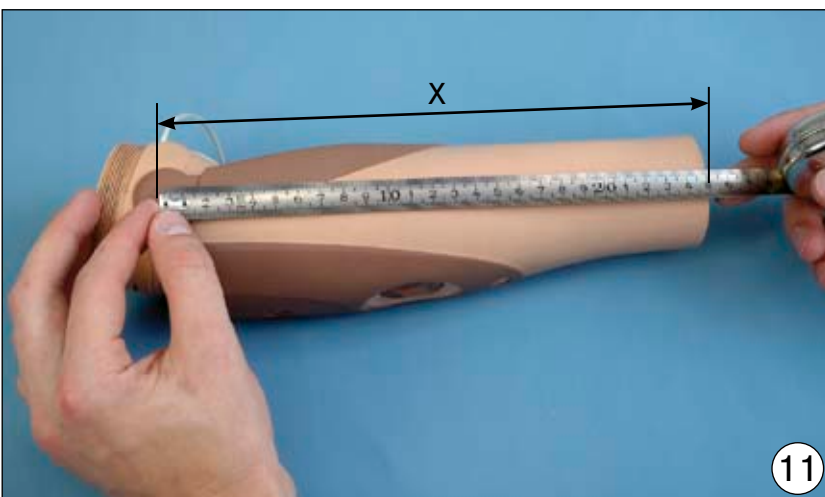Determine the length. Measure the distance between the front edge of the socket and the rotation center of the joint. Document value 'X'.

## 7 Length Ranges

### 7.1 Length Range 189 mm – 211 mm

Required accessories (see also section 5):


- Coaxial plug (item 5)
- Threaded tube, short, with lamination ring and counter nut (item 3)
- Rental DynamicArm® (item 8)

### 7.2 Length Range 214 mm – 256 mm

Required accessories (see also section 5):

- Retaining ring (item 7)
- Electric wrist rotator (item 6) / coaxial plug (item 5)
- Adapter short (item 2)
- Threaded tube, long, with lamination ring and counter nut (item 4)
- Rental DynamicArm® (item 8)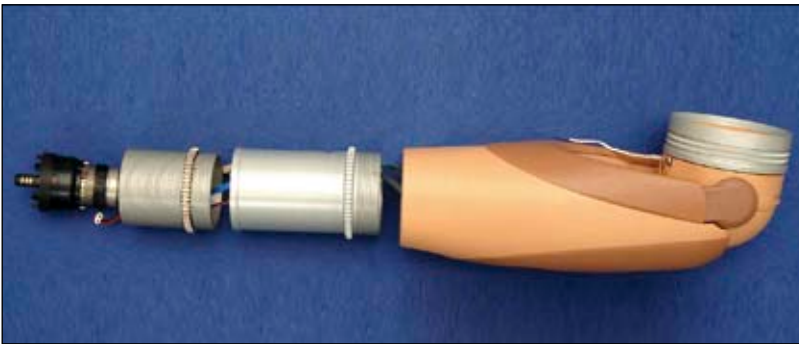


### 7.3 Length Range 259 mm – 305 mm

Required accessories (see also section 5):

- Retaining ring (item 7)
- Electric wrist rotator (item 6) / coaxial plug (item 5)
- Adapter long (item 1)
- Threaded tube, long, with lamination ring and counter nut (item 4)
- Rental DynamicArm® (item 8)



## 8 Length Adjustment of the Rental DynamicArm®

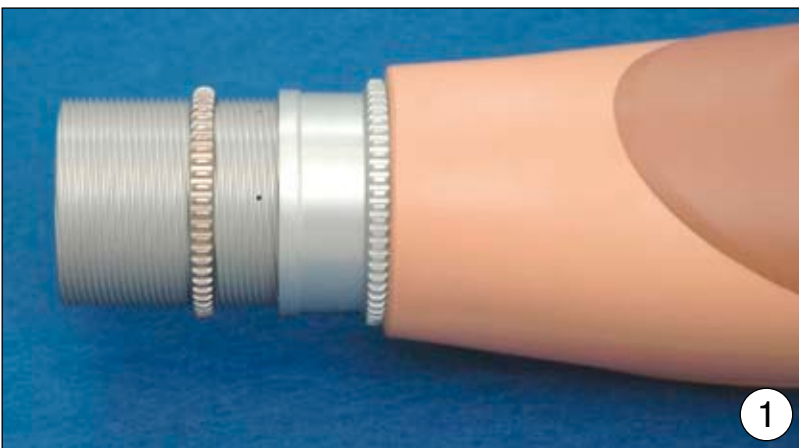1. On the basis of the measured length, you assemble the corresponding version.

2. Screw the threaded tube in or out as required and adjust the measurement 'X' (see section 6).

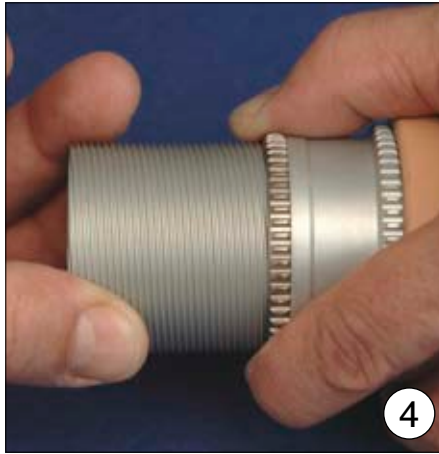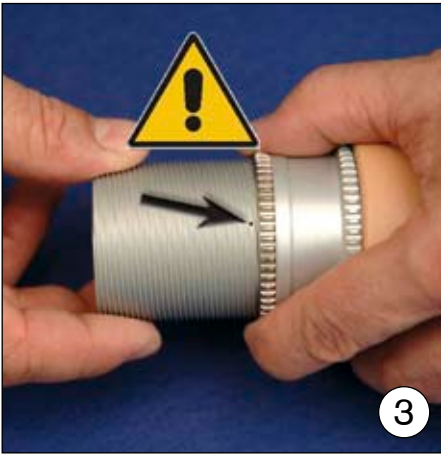3.

**Attention:** The threaded tube must be screwed in at least to the point where the marker bore holes (arrow) are covered by the counter nut. If this is not possible, the next longer version will have to be chosen.

4. Tighten the adapter and counter nut of the threaded tube clockwise (hand-tight).


5. Clamp the correctly positioned strap wrench into the vise.  
Fix the Rental DynamicArm® in the strap wrench.

First apply the hook wrench to the adapter (optional) and finally to the counter nut of the threaded tube. Pull the nut tight (clockwise) until the torque moment is reached.

5.1 The torque moment of 16 Nm will be reached as soon as the handles touch each other at the hook wrench.


## 9 Completion of the Rental DynamicArm®

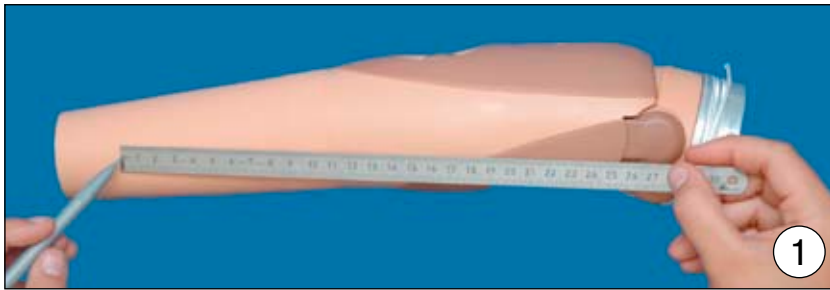
Slip the disposable foam cover over the Rental DynamicArm®.

Mark the corresponding length according to the determined measurement 'X' (see section 6, item 7).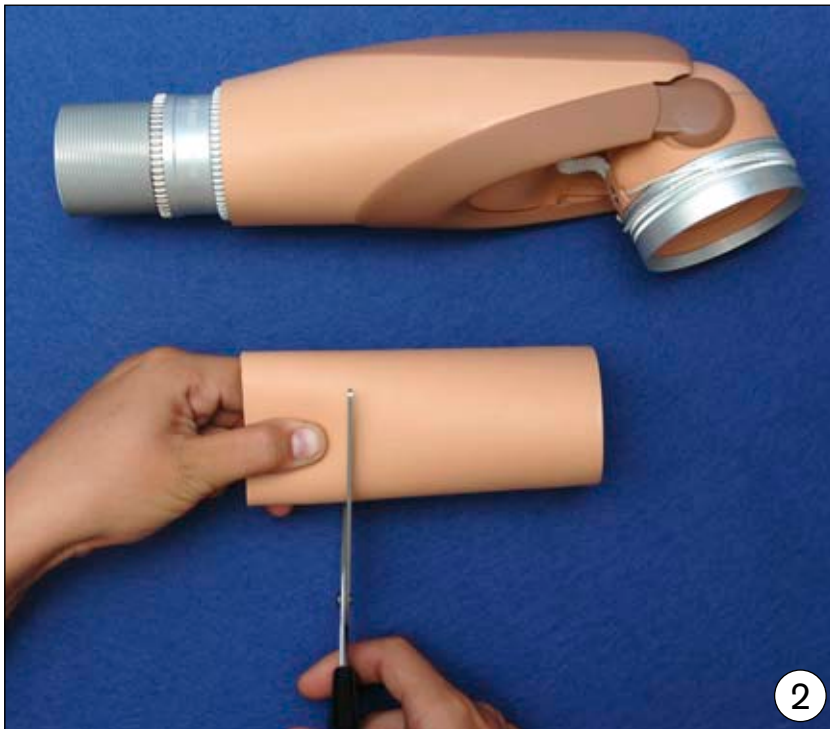

Use scissors to cut-off the disposable foam cover.

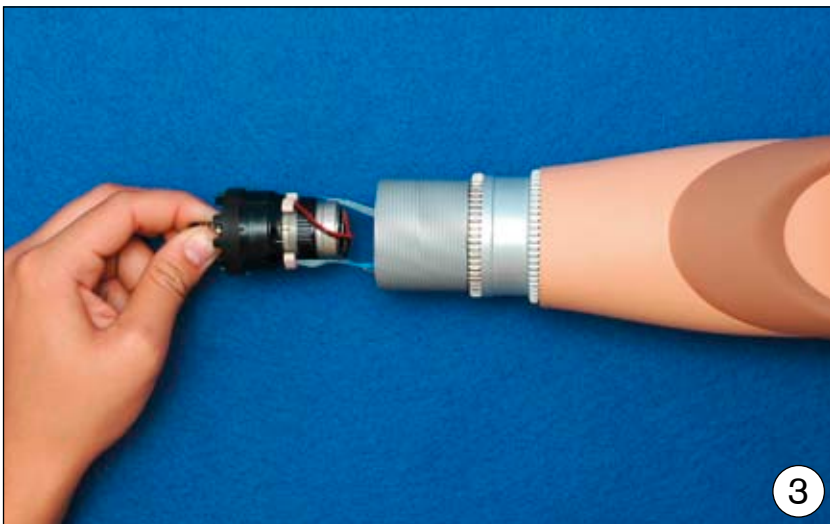Connect the cables with the electric wrist rotator / coaxial plug.

Next mount the electric rotator / coaxial plug into the lamination ring of the threaded tube. See the 647G152 Instructions for Use.

Pull the disposable foam cover previously cut to length over the Rental DynamicArm®.

Mount the prosthetic components.

Pull the cosmetic glove over the system electric hand.

In case of a fitting with the electric wrist rotator, make sure that the rotational movement is working smoothly.

## 10 Assemble the Rental DynamicArm®

After adaptation of the 12K100=RENTAL Rental DynamicArm®, you can fit the prosthesis with the prosthetic socket.

1. Dismantle the lamination ring of the 12K100=RENTAL as well as the prosthetic socket of the 12K100 DynamicArm® (see section 6, items 4 – 7).
2. Screw the socket onto the elbow joint. Or turn the friction screw until the socket can be screwed on the elbow joint (see 647G152 Instructions for Use).
3. Remove the Allen head screw. Then replace the red assembly clamp with the band clamp and reinstall the Allen head screws (see 647G152 Instructions for Use).
4. Turn the friction screw to set the friction of humeral rotation (see 647G152 Instructions for Use).
5. Adjust compensation to the patient by turning the finger wheel (see 647G152 Instructions for Use).

## 11 Technical Data

Power supply:	Li-Ion battery 1800 mAh
Maximum speed *):	270° / sec
Max. flexion angle:	15° – 145°

\*) Minimum lifting time is 0.5 sec, depending on the length of the forearm, on the terminal device used and the optimal compensation with AFB.

## 12 Liability

Otto Bock Healthcare Products GmbH, hereinafter called manufacturer, assumes liability only if the user observes the product's specified fabrication and use instructions, adheres to its care instructions, and follows its service schedule. This product may only be used in combination with components that have been authorized by the manufacturer (please refer to the instructions for use and catalogs). The manufacturer does not assume liability for damage caused by component combinations and applications that were not authorized by the manufacturer.

This product may only be opened and repaired by authorized Otto Bock staff.

## 13 CE conformity

This product meets the requirements of the 93/42/EWG guidelines for medical products. This product has been classified as a class I product according to the classification criteria outlined in appendix IX of the guidelines. The declaration of conformity was therefore created by Otto Bock with sole responsibility according to appendix VII of the guidelines.

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